

## Hydro Queen™ Superficial Peeling

The Newest Skin Resurfacing

Non-Invasive

Painless

Simultaneous Actions

"Cleansing"

"Hydration"

"Extraction"

"Shining"

### Technical specification

Ultrasound	1.0 MHZ
Motor	upto 600mmHg (±10%)
Power consumption	80Watt
Input Voltage	220VAC / 50 / 60Hz
Size	350(D) X 350(L) X 1000 (H) mm
Weight	30Kg
Aqua Handpiece	Face(Regular Size) Nose(Narrow Size)
Aqua Waste tank	1200cc

Manufacturer : **IDS**

Address: #105, Byok san 3 cha Digital valley 212-13, Guro-3Dong, Guro-Gu, Seoul Korea  
Tel : 82-2-2025-1151 Fax : 82-2-2025-1154 E-mail : [info@ids.co.kr](mailto:info@ids.co.kr)

## Refreshing energy\*

Will make your skin and scalp much cleaner and whiter.

### Aqua Therapy System Hydro Queen™

#### Removal of horny substances and supply of moisture at the same time\*

Hydro Queen™ is designed to promote exfoliation to remove aged dead skin cells and sebum and, at the same time, inject highly concentrated, nutritious serum so as to make skin and scalp moist, smooth, and lustrous.

#### Safety: neither downtime nor side-effects\*

Because treatment does not cause any damage to skin and scalp, patients can lead a normal life right after treatment. There are no side effects such as injuries, pain, burns, rashes, etc.

#### Multiple skincare effects\*

Treatment removes foreign materials (such as cosmetics and sebum) so users can expect to achieve whitening and pore-tightening effects.



All the more clean and bright with refreshing energy in our skin and scalp!!!


## Hydro Queen™ is ...

not a previously available decortication, but a new-concept natural spa skincare system, which enables you to painlessly achieve in a comfortable environment a variety of treatment ranging from cleansing of skin to removal of obsolescent horny substances to extrusion of sebum to supply of nutrition simultaneously. You will feel a moist and refreshing sensation after treatment. Ultra Sound probe can for treat pain release and fat treatment.

## User Manual for Hydro Queen™

Hydro Queen™ adopts a vacuum extrusion method. The hand piece should come into close contact with the skin. You can control pressure for both fine bubbles and serum liquid to perform treatment. Red areas on the skin that may be created during operation will disappear normally in 20-30 minutes and the skin restores healthy and lustrous condition.

## Hydro Queen™ Name and function of parts and components

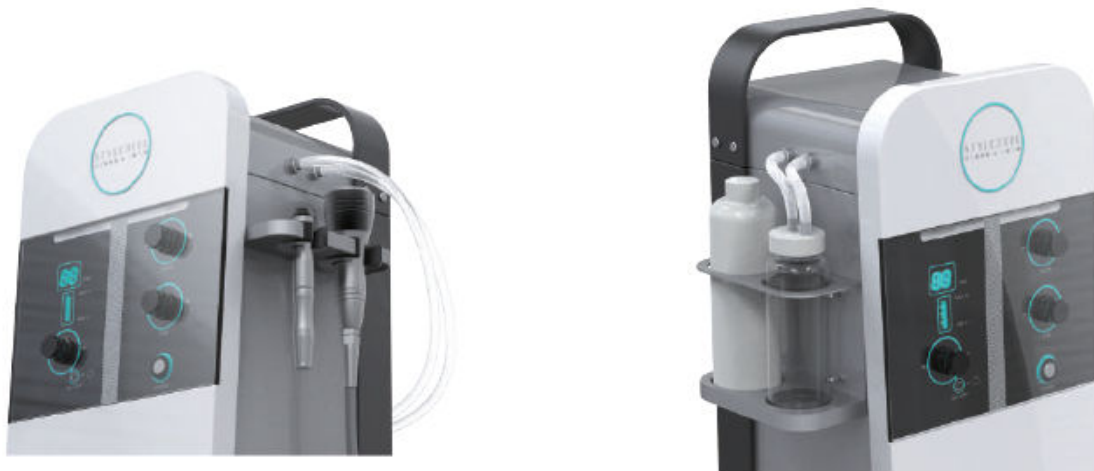

### 01 Cleansing

wipe wastes 2-3 times off the surface of the face with a cleansing cream

### 02 Diagnosis of scalp and skin and Selection of serum

identify whether the skin is normal, oily, sensitive, or of acne type and make selection of a serum suitable for the skin type.  
[AG serum: for normal skin; BG serum: for acne or sensitive skin]

### 03 Treatment (removal of keratin and sebum)

first spread the selected fine bubble and serum over the face and then divide the face into 3-5 areas to start treatment for about 2 minutes.

### 04 Washing

wash and wipe out the face cleanly with a spongy after treatment.

### 05 Treatment (supply of moisture and nutrients) and finish

perform treatment while letting AO serum penetrate into the skin; after treatment, give a finishing touch with a skin balm or a nourishing cream.



In the order of Neck ►►► Chin ►►► Cheek ►►► Forehead ►►► Nose

- 1, Neck: after turning face a little sideways, make lymph nodes relax by massaging the area from top to bottom and in the opposite way with the sternocleidomastoid region in the center.
- 2, Nose: perform treatments repeatedly because there is lots of sebum in this region.

- 3, Perform treatment by traveling all across the face from bottom to top along the texture of muscles.
- 4, Treatment is recommended to be made for about 2 minutes respectively in the order of the neck → chin → cheek → forehead → nose.

## Special tips

### For removal of keratin as well as for extrusion



Whole face



Around the nose

The special tip of the Hydro Queen™ consists of two suction holes and one serum sidcharge hole. Its specially-designed spiral tip is intended to hold a product long enough to soften dead cells of the scalp and skin, provide nutrients, and absorb them in a vacuum state to maximize its peeling and extrusion effects.

## Hydro Queen™ Serum



SOLUTION

LASER DOCTOR

### MBPE-1 [For all Skin Type-For the general skin, 550ml]

Since glucosamine and lactic acids promote the peeling of the skin to remove impure substances softly and plant extracts soothe and moisten the skin smoothly, the product enables a scientific treatment that does not produce any injuries or rashes while achieve a bright and healthy-looking skin regeneration.

### Laser Doctor [Nutrition and supply of moisture for all type of skin, 490ml]

The deep penetration of vitamin C, E, Aloe Vera, and green tea extract will help skin to restore and regenerate damaged skin cells to maintain a moist and bright skin tone without any side effect.