


Specifications

Handles Number	4 handles (1 Large, 1 Medium, 1 Small, 1 Special for Chin-Option)
Handle screen	5 inch touch screen
Vacuum intensity	0-100kPa adjustable
Handle sizes	Large : 220*76*125mm; Medium : 160*56*65mm; Small : 125*45*70mm
Machine screen	10.4 inch touch screen
Cooling temperature	-15°C--5°C
Cooling system	Water, Air, Semi-conductor triple cooling system
Input power	2500W
Output power	1600W
Machine Dimensions	1230*450*420mm
Fuse specifications	Ø5x25 15A
Machine Dimensions	AC220V±10%,10A,50HZ/ AC110V±10%10A,60HZ

ORCHESTRA *Cool*

Health & Beauty Care System

Smart Mode

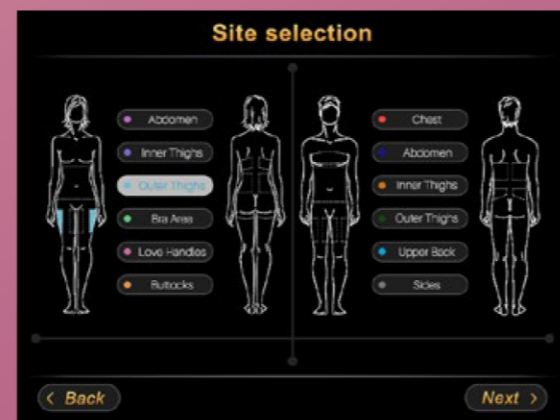

Cool

ORCHESTRA

Health & Beauty Care System

Fat treatment & Cryotherapy

Abdomen
Thigh
Chest
Love Handle
Buttock
Upper Back
Arm
Leg


ids

Address 194, Simgung-ro, Gwangtan-myeon, Paju-si, Gyeonggi-do, Republic of Korea
Tel +82-2-2025-1150~3 Fax +82-2-2025-1154 E-mail info@ids.co.kr

Working principle


As triglyceride in fats will be converted into solid in particular low temperature, this machine uses precisely controlled cooling energy to the target areas for treatment. It selectively targets at certain fat bulges and eliminate fat cells through a gradual process that does not harm the surrounding tissues and reduce unwanted fat, When fat cells are exposed to precisely control cooling, the triglyceride in fats will be converted into solid, and they will be Crystallization, aging, and then die. The dead fat cells will be out of body by natural metabolism.

Treatment Process

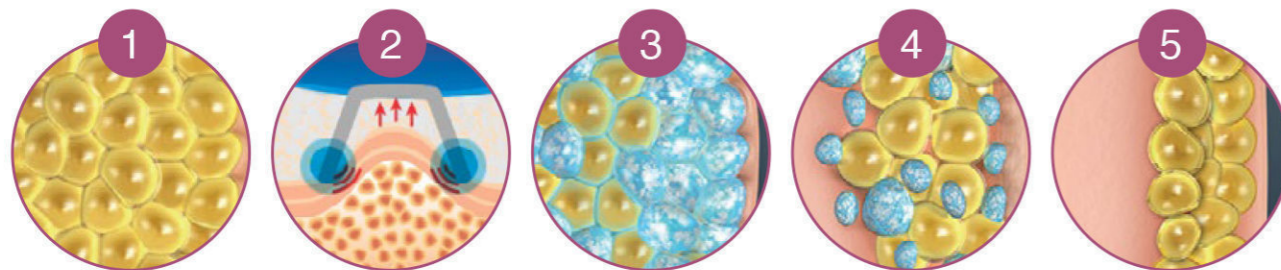

- 01 | Treatment area prior to applying cooling
- 02 | Precisely controlled cooling is applied to the treatment area
- 03 | Underlying fat cells are targeted.
- 04 | Fat cells begin a natural removal process that continues for several months
- 05 | The natural removal of fat cells over time results in gradual fat reduction

Dynamic handpiece

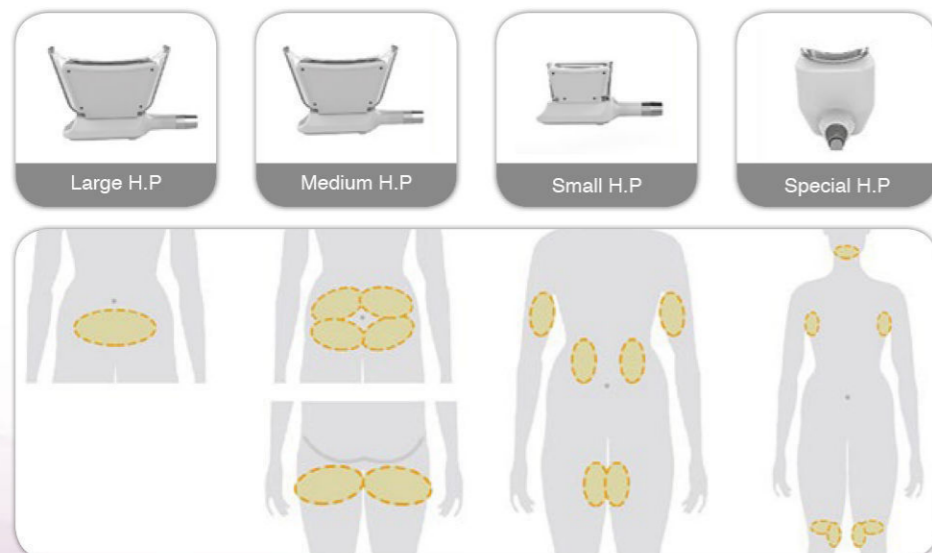

- Four different sizes handpieces optional, easy treatment for whole body.
- Four handpieces working simultaneously, save staff cost, save treatment time.
- Unique double chin treatment handpiece, lead the latest technology in the world.
- Super strong cooling system ensure the minimum treatment temperature reduce to -15 → 5°C
- Non-surgery, no risk, easily intelligentize and humanization operation
- No need recovery, no downtime, come back to work immediately.
- Variety accessories for multi-functional treatment
- Luxury design
- Easy operation for all program

